



Step 1:
Remove funnel and tube from urine transport kit.



Step 2:
Unscrew tube cap.



Step 3:
Firmly screw funnel onto tube (keep tube upright to prevent spilling the liquid preservative).



Step 4:
Point the spout towards the toilet and begin urinating into the funnel.



Step 5:
Continue urinating until bladder is emptied.



Step 6:
Unscrew funnel from tube (keep tube upright to prevent spilling the urine sample).



Step 7:
Firmly screw cap onto the tube (keep tube upright to prevent spilling the liquid preservative).



Step 8:
Dispose of the funnel.



Step 9:
Gently invert UrNCollect Tube five times to mix the contents.

Step 10:
Place (1) barcode label on UrNCollect Tube.
Place (1) barcode label on Test Requisition Form.